

Morning Check List

Phonak Campus SX & MicroMLxS/MLxS

Evening Check List

- 1 CONNECT** up the system as it will be used & **TURN ON.**

Remember to check that plugs are pushed home.

- 2 INVESTIGATE** any reported operating problems from yesterday.

Intermittent sound problems can be difficult to trace, ask the user to explain what happened when.



- 3 TEST** the overall system.

Give the transmitter to a friend or place it somewhere a few metres away where it will pick up some identifiable sound and listen to the output of the hearing aid.

If you are the hearing aid user, make sure that you can hear a clear sound from the transmitter. If you are a hearing helper, use a stetoclip and attenuator to listen to the hearing aid.



- 4 RECEIVER** Gently check the connections between the MLxS receiver, the shoe and hearing aid to check that the sound does not break up or crackle.

Worn hearing aid contacts are a common source of problems, if necessary return the hearing aid to the clinic for re-casing.



- 5 TRANSMITTER** While still listening to the hearing aid, ask your helper to gently wiggle the mic lead on the Campus SX transmitter make sure that the sound does not break up or crackle. Change any suspect items.

- 6 BATTERY STATUS** Use a battery tester to check the condition of the hearing aid battery. Look at the Campus S display to see that there is enough power for the time that the system will be used - but of course, you did check & recharge last night!



- 7 RE-ORDER** spares as required - always keep spare shoes, microphone and hearing aid batteries etc.



Campus SX and lapel microphone





Hearing aid shoe and MLxS

- 1** The Campus SX has a battery life of approx 12 hours so we advise charging daily.

- 2** Check that the Campus SX unit on charge has scrolling battery bars. If there are no scrolling bars then the unit is not charging - check the connections.

- 3** Do not waste time unnecessarily disconnecting items. You will only have to put them all back together in the morning and it saves general wear and tear.

- 4** As the MLxS receiver takes its power from the hearing aid battery it is very important that the battery is in good condition. A low hearing aid battery will result in increased hiss and reduced range - the normal life of a hearing aid battery will be more than halved when used with an MLxS. If in doubt, fit a new battery in the hearing aid.

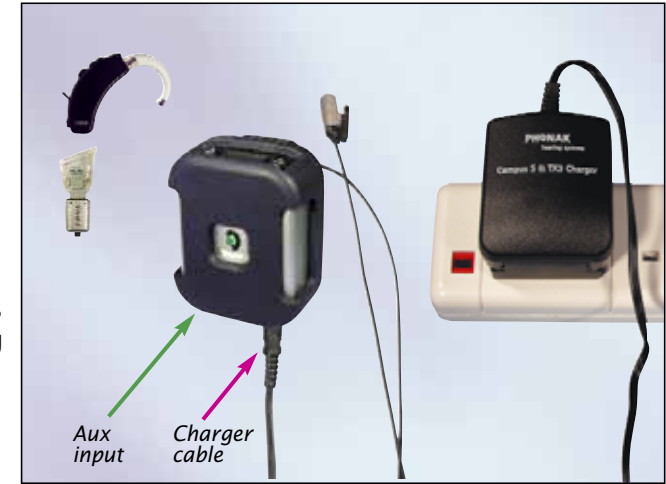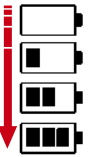

On/off (long press)
Mic mute (short press)


Control buttons locked

Channel UP/DOWN

MLxS channel change (synchronization)

LCD display with channel and battery status

In case of difficulty, consult the Campus SX User Guide which contains much useful help.

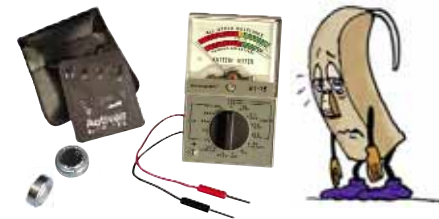

To reprogramme the Campus SX for one channel only, visit www.connevans.com/phonak/tx

Press these buttons simultaneously for 2 secs to lock



BCAMMLSCK 2.0